

... and around itself.
Zetuvit® Plus – can be combined with a lot of wound dressings from HARTMANN.



PAUL HARTMANN AG
D-89522 Heidenheim
Germany

Would you like to know more about HARTMANN?
Visit our Website: www.hartmann.info



NEW!

The powerful response to heavily exuding wounds.
New Zetuvit® Plus.
Unique, absorbent, without compromise.

For example:



Atrauman® Ag

The silver-containing impregnated dressing is particularly suitable in combination with Zetuvit Plus in heavily exuding or infected wounds. In contact with wound fluid, Atrauman Ag provides for safe germ destruction and keeps at the same time the wound margins soft and supple. It can be cut to size and thus guarantees for an individual fit to the wound.



Sorbalgon®

If the wound is exuding severely and is fissured or forms pockets, a combination of Zetuvit Plus and Sorbalgon suggests itself. Sorbalgon is a dressing made of calcium alginate fibers which form a gel in contact with wound fluid. For very deep wounds Sorbalgon is available as Sorbalgon T tamponade ribbons.



Presentations	Size	Unit	Code No.	Units per Case
Zetuvit® Plus	10 x 10 cm	Pack of 10	413 710	8
	10 x 20 cm	Pack of 10	413 711	6
	15 x 20 cm	Pack of 10	413 712	8
	20 x 25 cm	Pack of 10	413 713	6
	20 x 40 cm	Pack of 5	413 714	8

P316 (06/07)



The absorption capacity it has in itself ...

Extra absorbent: The combined absorbent dressing pad Zetuvit® Plus.



Sometimes it simply has to be more.

Up to now there were only compromises for heavily exuding wounds, and no ideal solutions. The consequences were frequent dressing changes which were very time consuming and caused high costs.

Sometimes it has to be Zetuvit Plus.

New – the unique combination of absorption capacity, softness and padding: Zetuvit Plus from HARTMANN. The absorbent dressing pad with its special four-layer composition and the extremely high absorption capacity is excellently suitable for heavily exuding wounds. Convince yourself:

Two-layer outer non-woven.

It is particularly soft and surrounds the entire product. The hydrophobic outer surface of the non-woven prevents it from sticking to the wound. The hydrophilic cellulose fibers of the inner surface of the non-woven have a high capillary activity and pass exudate quickly in the absorbent core.

Superabsorbent core.

The cellulose fluff in the absorbent core with a liquid-retaining super absorber bind and trap exudate. In addition, the soft properties of the absorbent core provide for a pleasant padding effect.


Air-permeable special non-woven.

On the side facing away from the wound, the product features a special water-repellent non-woven that is permeable to air. This side is marked in green to ensure the correct application of the dressing.

Thin non-woven fabric.

It entirely surrounds the core and uniformly distributes the liquid.

Your additional benefits:

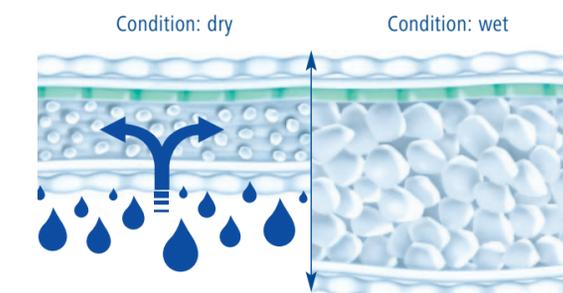

Optimal absorption capacity.

Zetuvit Plus absorbs more than double the volume of conventional absorbent dressing pads and is thus ideal for heavily exuding wounds. The exudate is trapped, so that Zetuvit Plus can also be applied under compression therapy.

High protection of the wound.

Zetuvit Plus is very pleasant on the skin. It is made of a skin-friendly, air-permeable and soft material and has no sharp edges. The patient is treated comfortably and safely.

Promoting wound healing.

The hydrophobic outer non-woven reduces the tendency of adhesion to the wound. In addition, the excessive exudate is passed into the core quickly which prevents accumulation of wetness on the one hand and reduces the risk of maceration and infection on the other hand. The wound healing process can progress. On account of the high absorption capacity of Zetuvit Plus the frequency of dressing changes is reduced which promotes the rest of the wound.

Considerable time and cost saving.

The clear marking of the side facing away from the wound permits fast and easy application to the wound. On account of the high absorption capacity, the intervals of dressing changes can be reduced which ensure an economic work process. Thus, Zetuvit Plus is an economically sensible solution when it comes to treating heavily exuding wounds in an ideal way.