

HydroTherapy

Case studies

Case 1: Patient, female, 79 years old

Description of the wound: 3.5 cm wide Ulcus cruris venosum above left ankle joint, previous wound treatment: for the last four weeks with Betaisodona



1. 15.05.14
Start with HydroTherapy:
HydroClean® plus, 4 cm, round,
accompanying compression therapy

2. 22.05.14
Continuation of treatment with
HydroClean® plus. Change on
05.06. to HydroTac®, since majority
of the wound had granulated.



3. 16.06.14
Wound closed up


(07/14) 086 2567

HydroTherapy

Case studies

Case 2: Patient, male, 78 years old

Description of wound: A small infectious ulceration, on right medial lower leg, treatment started with HydroClean® plus, 4 cm, round, followed by HydroTac® 10×10 cm, fixing with Peha-haft®, observation period: 31.10.2013–26.11.2013

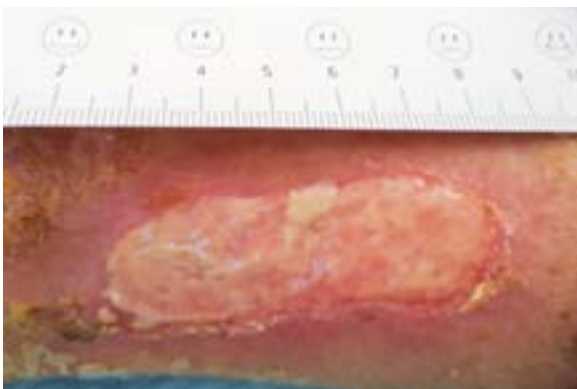

1. 31.10.2013
HydroClean® plus 4 cm round



2. 08.11.2013
HydroClean® is removed and
changeover to HydroTac® is
carried out.



3. 26.11.2013
The aim of the therapy was
achieved after changing dressing
seven times.

HydroTherapy

Case studies

Case 3: Patient, male, 82 years old

Description of wound: Post-operative wounds after cholecystectomy, 14-day treatment with Alginat and Urgo Start, no improvement of wound situation



1. 21.05.14
Start HydroTherapy: three wounds with HydroClean® plus, 4 cm, round: more wounds treated with HydroClean® plus 5.5 cm round



2. 26.05.14
Complete healing of one wound: continuation of treatment with HydroClean® plus

3. 02.06.14
Switchover to HydroTac®



4. 18.06.14
Wound completely healed

(07/14) 086 2567



HydroTherapy
Efficacy. And Simplicity.



HydroTherapy

Case studies

Case 4: Patient, female, 85 years old

Description of wound: Abrasion on left-arm elbow, at the beginning approx. 1.7 cm long wound; increased to 4 cm; previous treatment with Urgo Tüll



1. 13.06.14
Start HydroTherapy: treatment with HydroClean® plus, 4 cm, round



2. 19.06.14
Until then, only one change of dressing HydroClean® plus; change to HydroTac® 8×8 cm

3. 25.06.14
Complete wound closure, before reaching this point only one change of dressing HydroTac® required

(07/14) 086 2567