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Produktbeschreibung

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Instructions for use

Product description

RespoSorb Super is a wound dressing used on severely exuding wounds. The absorbent core binds and traps exudate.

Composition

RespoSorb Super is a combined absorbent dressing pad which consists of four layers of different materials. On the wound side, the product features a soft, white, hydrophilic nonwoven (viscose and polyamide) which prevents the dressing from sticking to the wound. The inner dressing core consists of soft cellulose fluff blended with liquid-retaining polyacrylate polymers. This absorbent core is enclosed in a thin nonwoven fabric that uniformly distributes the liquid. On the side facing away from the wound, the product features a green layer of polypropylene nonwoven, which is water-repellent but permeable to air and allows gas exchanges.

Properties and mode of action

RespoSorb Super rapidly absorbs exudate and reliably binds it within the absorbent core. Exudate removal eliminates inhibitory factors from the wound, such as e.g. proteases, thus enabling speedy wound healing. The increased absorption capacity of RespoSorb Super reduces the frequency of dressing changes, and this promotes wound rest and affords additional protection against contamination.

Apart from its highly absorbent quality, RespoSorb Super also has an effective padding effect.

Indications

RespoSorb Super is especially suitable for the treatment of superficial, severely exuding acute or chronic wounds (such as e.g. decubitus, leg ulcers, exulcerating tumours, etc.).

Mode of application

Select RespoSorb Super to match the wound size so that the dressing extends at least 1–2 centimetres beyond the wound margins. The wound dressing cannot be cut to size. Place RespoSorb Super with the white side on the wound so that the green special nonwoven layer faces away from the wound. Secure the dressing e.g. with adhesive tape, conforming bandages or, if necessary, compression bandages. Change the dressing if clinically indicated or when exudate reaches the edges of the dressing or is noticeable through the green nonwoven top layer.

In infected wounds a combination with Atrauman Ag is possible.

With decreasing exudate, the use of a suitable hydroactive wound dressing (such as e.g. HydroTac) is recommended.

Contraindications

RespoSorb Super on dry wounds or on exposed bones, muscles or tendons. Do not use RespoSorb Super on patients who may be allergic to any of its ingredients.

Special precautions

Medical assessment of the wound condition and the causes of wound-healing impairment is necessary before treating wounds with an impaired healing tendency. Treatment with RespoSorb Super cannot replace a causal treatment of the wound-healing impairment. The sealed seam of the peel pack contains natural rubber latex, which may cause allergic reactions!

General instructions

Contents sterile unless peel packages are damaged. Protect from sunlight. Store flat in a dry place. Keep out of the reach of children.

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