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RespOsil Silicone Border ist eine sterile single-use self-adhesive absorbent dressing with a silicone interface for long-term treatment of acute and chronic exuding wounds. It is used on adults in clinical or homecare environments and can be combined with cleansing solutions and local antiseptics, primary dressings and additional fixation. It can be used in context of compression therapy. RespOsil Silicone Border can be used by healthcare professionals and lay persons under the supervision of healthcare professionals.

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RespOsil Silicone Border is suitable for the treatment of a wide range of exuding wounds: acute wounds (burns, post-op, trauma wounds), chronic wounds (venous ulcers, pressure ulcers, diabetic foot ulcers, tumour wounds).

### Mode of application

- Cleanse the wound according to clinical practice. Make sure that the surrounding skin is dry and clean to ensure that the dressing sticks.
- Select the dressing's size and shape, so that the green absorbent pad exceeds the wound edges by approx. 1 to 2 cm.

- Do not cut the green absorbent pad. The transparent border can be cut to adapt the dressing to specific body areas.
- Remove the protective sheets according to the application instructions. Apply the adherent side to the wound with the green side facing away.
- Press down the adhesive edges firmly and avoid wrinkles in the border to ensure the dressing stays position in place.
- No additional fixation is required. If indicated, compression bandages can be applied over the dressing.

The dressing can remain on the wound for up to 7 days, depending on the condition of the wound and the clinician's instructions.

- The dressing must be changed if clinically indicated or when exudate approached the rim of the green absorbent pad.
- Do not cut the green absorbent pad. The transparent border can be cut to adapt the dressing to specific body areas.
- Remove the protective sheets according to the application instructions. Apply the adherent side to the wound with the green side facing away.
- Press down the adhesive edges firmly and avoid wrinkles in the border to ensure the dressing stays position in place.
- No additional fixation is required. If indicated, compression bandages can be applied over the dressing.
- The dressing can remain on the wound for up to 7 days, depending on the condition of the wound and the clinician's instructions.
- The dressing must be changed if clinically indicated or when exudate approached the rim of the green absorbent pad.

### Contraindications

Do not use RespOsil Silicone Border in case of hypersensitivity to any of its components. RespOsil Silicone Border is not recommended for use on the following skin conditions:

- Do not cut the green absorbent pad. The transparent border can be cut to adapt the dressing to specific body areas.
- Before treating a wound that is not healing as expected, consult a clinician. Treatment with RespOsil Silicone Border cannot replace a causal treatment of the wound–healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before this dressing can be used.
- In all cases, follow the clinician's instructions.
- In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data on the use of this population groups this dressing should be used with caution and following a professional's recommendation.
- It is not recommended to stand on the dressing.
- Before opening the product packaging, inspect carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.
- Resuing a single-use medical device is dangerous. Recrossing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

### Side effects

If any of the following side effects occur, consult a clinician: increased exudate, Patient discomfort, itching, redness, rash, contact dermatitis, adherence of the dressing to the wound, itching, burning, oedema, itchy blisters, multiple ulceration, skin tears, blisters, erythema.

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### IT instructions for use

### Product description

RespOsil Silicone Border is a sterile self-adhesive absorbent dressing with a silicone interface. The dressing comprises a perforated silicone wound contact layer, a semipermeable polyurethane backing film and a green absorbent pad in between.

those two layers. Acrylic adhesive is used to bind these layers together. The backing film is permeable to air but waterproof, which allows the patient to shower. The silicone layer facilitates the application of the dressing and promotes non-adherence to the wound. It also allows the patient to take a shower. The adhesive consists of a composite of acrylic, polyurethane and polypropylene. The absorbent pad comprises cellulose and superabsorbent hydrocolloid polyurethane. Trägerfolien (Hydrophobes und Polypropylen) ausgetüschelt. Das Saugkissen besteht aus Zellulose und superabsorbierendem Polyacrylat (SAP), um Exsudat aufzunehmen und zu binden, und ist mit hydrophilem Zellulosegewebe umhüllt, das Exsudat verteilt. Es ist mit einem grünen Hydrophobes Polypropylen-Vlies bedeckt, das die verbindende Seite kennzeichnet.

**Propriétés et mode of action**
Le produit est stérile et adhère et retient un haut niveau d'exsudats in the absorbent pad and the dressing is able to redistribute pressure. It features a micro-adherent silicone interface and adhesive borders, so no additional materials are required to secure the dressing. The backing film is bacteria and showerproof. The dressing changes can be carried out atraumatically and almost painlessly.

**Intended purpose**
RespOsil Silicone Border is a sterile single-use self-adhesive absorbent dressing with a silicone interface for long-term treatment of acute and chronic exuding wounds. It is used on adults in clinical or homecare environments and can be combined with cleansing solutions and local antiseptics, primary dressings and additional fixation. It can be used in context of compression therapy. RespOsil Silicone Border can be used by healthcare professionals and lay persons under the supervision of healthcare professionals.

### Indications

RespOsil Silicone Border is suitable for the treatment of a wide range of exuding wounds: acute wounds (burns, post-op, trauma wounds), chronic wounds (venous ulcers, pressure ulcers, diabetic foot ulcers, tumour wounds).

### Mode of application

- Cleanse the wound according to clinical practice. Make sure that the surrounding skin is dry and clean to ensure that the dressing sticks.
- Select the dressing's size and shape, so that the green absorbent pad exceeds the wound edges by approx. 1 to 2 cm.

- Do not cut the green absorbent pad. The transparent border can be cut to adapt the dressing to specific body areas.
- Remove the protective sheets according to the application instructions. Apply the adherent side to the wound with the green side facing away.
- Press down the adhesive edges firmly and avoid wrinkles in the border to ensure the dressing stays position in place.
- No additional fixation is required. If indicated, compression bandages can be applied over the dressing.

The dressing can remain on the wound for up to 7 days, depending on the condition of the wound and the clinician's instructions.

The dressing must be changed if clinically indicated or when exudate approached the rim of the green absorbent pad.

### Contraindications

Do not use RespOsil Silicone Border in case of hypersensitivity to any of its components.

### Special precautions

- Do not cut the green absorbent pad. The transparent border can be cut to adapt the dressing to specific body areas.
- Before treating a wound that is not healing as expected, consult a clinician. Treatment with RespOsil Silicone Border cannot replace a causal treatment of the wound–healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before this dressing can be used.
- In all cases, follow the clinician's instructions.
- In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data on the use of this population groups this dressing should be used with caution and following a professional's recommendation.
- It is not recommended to stand on the dressing.
- Before opening the product packaging, inspect carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.
- Resuing a single-use medical device is dangerous. Recrossing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

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### Side effects

If any of the following side effects occur, consult a clinician: increased exudate, Patient discomfort, itching, redness, rash, contact dermatitis, adherence of the dressing to the wound, itching, burning, oedema, itchy blisters, multiple ulceration, skin tears, blisters, erythema.

### Incident reporting

Für Patienten/Anwender/Dritte in der Europäischen Union und in Ländern mit identikal regulatory regime (Regulation (EU) 2017/745 on Medical Devices), if during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

### Product disposal

To minimize the risk of potential infection hazards, or environmental pollution, dispose components of the medical device should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards. Dispose of the medical device with household waste.

### Date of revision of the text:

2023-09-28

AU – PAUL HARTMANN Pty. Ltd. · Macquarie Park, NSW 2113

GB – PAUL HARTMANN Ltd. · Heywood Distribution Park, Pilsworth Road, OL10 2TJ

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